
**OPERATOR'S MANUAL - ENGLISH
MANUAL DE INSTRUCCIONES -
DOMESTIC SPANISH**

PLATINUM 5
HFII

**OXYGEN CONCENTRATORS STANDARD,
WITH/SENSO₂[®] AND HOMEFILL™ II
CONCENTRADORES DE OXÍGENO
STANDARD, CON/SENSO₂[®] Y HOMEFILL™ II**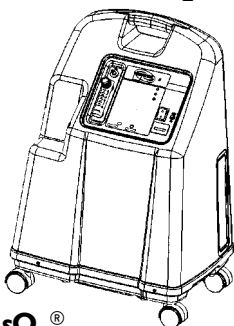

**Invacare 5 ^{with/Senso}O₂[®]
Model IRC5LXO2**

**Invacare 5 ^{con/Senso}O₂[®]
Modelo IRC5LXO2**

**Invacare 5
Model IRC5LX**

**Invacare 5
Modelo IRC5LX**

DEALER: THIS MANUAL MUST BE GIVEN TO THE END USER.

USER: BEFORE USING THIS CONCENTRATOR, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.

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Yes, you can.®

SPECIAL NOTES

WARNING/CAUTION notices used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

SAVE THESE INSTRUCTIONS

TABLE OF CONTENTS

SPECIAL NOTES	2
SAFETY SUMMARY	4
FEATURES	7
SPECIFICATIONS	8
HANDLING	11
Unpacking	11
Inspection	11
Storage	11
INTRODUCTION	12
OPERATING INSTRUCTIONS	12
Select a Location	12
Recommended End-User Guidelines for Optimum Performance	12
Plug in Power Cord	13
Connect Humidifier (if so prescribed)	13
Oxygen Tubing	14
Power Switch	15
Flowrate	15
SensO ₂ [®] Oxygen Purity Indicator- If your Unit Features the O ₂ Sensor	15
Initial Startup of the Concentrator	16
Explanation of Indicator Lights	16
Oxygen Purity Indicator- If your Unit Does Not Features the O ₂ Sensor	17
MAINTENANCE	17
Cleaning the Cabinet Filters	17
Cabinet	18
Humidifier	18
TROUBLESHOOTING GUIDE	19
PREVENTIVE MAINTENANCE RECORD	20
WARRANTY	21
SPANISH OWNER'S MANUAL (MANUAL DEL USARIO)	23

SAFETY SUMMARY

SAFETY SUMMARY

CAUTION

“Caution: Federal law restricts this device to sale or rental by or on order of a physician, or any other practitioner licensed by the law of the State in which he practices to use or order the use of this device.”

Risk of electric shock. **DO NOT** disassemble. Refer servicing to qualified service personnel.

READ THE FOLLOWING INFORMATION BEFORE OPERATING THIS PRODUCT.

WARNING

The use of oxygen therapy requires that special care be taken to reduce the risk of fire. Any materials that will burn in air, and some that will not, are easily ignited and burn rapidly in high concentrations of oxygen. For safety concerns, it is necessary that all sources of ignition be kept away from the product and preferably out of the room in which it is being used. **NO SMOKING** signs should be prominently displayed.

A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances **MUST** be kept away from the oxygen concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless recommended by Invacare.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes at a time. Shorter periods of operation may reduce maximum product life. If it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, call Qualified Service Personnel for examination and repair.

Keep the cord away from **HEATED** or **HOT** surfaces.

NEVER drop or insert any object into any opening.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like. Fill humidifier with water to the level shown by the manufacturer. **DO NOT** overfill.

WARNING (CONTINUED)

Keep at least 3 inches away from walls, draperies, furniture, and the like.

Invacare recommends that Crush-Proof oxygen tubing (supplied by Invacare) be used with this product and **NOT** exceed 50 ft. (15.2m) in length.

MAINTENANCE

The Invacare Oxygen Concentrator was specifically designed to minimize routine preventive maintenance at intervals of once per year. Only professionals of the healthcare field or persons fully conversant with this process such as authorized or factory trained personnel should perform preventive maintenance or performance adjustments on the oxygen concentrator.

RADIO FREQUENCY INTERFERENCE

Most electronic equipment is influenced by Radio Frequency Interference (RFI). **CAUTION** should be exercised with regard to the use of portable communications equipment in the area around such equipment.

TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS.

WARNING

ALWAYS unplug this product **IMMEDIATELY** after using. Avoid using while bathing. If continuous usage is required by the physician's prescription:

The concentrator must be located in another room at least seven (7) feet from the bath.

DO NOT come in contact with the concentrator while wet.

DO NOT place or store product where it can drop into water or other liquid.

DO NOT reach for product that has fallen into water. **UNPLUG IMMEDIATELY.**

A product should **NEVER** be left unattended when plugged in.

SAFETY SUMMARY

WARNING (CONTINUED)

This device is to be used only in accordance with the prescription of a physician and this Owners Manual. If at any time the patient or attendant conclude that the patient is receiving an insufficient amount of oxygen, the supplier and/or physician should be contacted immediately. No adjustments should be made to the flowrate unless prescribed by a physician.

Close supervision is necessary when this product is used near children or physically-challenged individuals.

Use this product for only intended use as described in this manual.

DO NOT use parts, accessories or adapters other than those authorized by Invacare.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

Use of some administration accessories or certain humidifiers not specified for use with oxygen concentrator may impair the performance.

In certain circumstances oxygen therapy can be hazardous. Invacare recommends that you seek medical advice before using this machine.

This equipment is to be used as an oxygen supplement and is not considered life supporting or life sustaining.


Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

POLARIZED PLUG INSTRUCTION


This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT ATTEMPT TO DEFEAT THIS SAFETY FEATURE.**

FEATURES

FRONT VIEW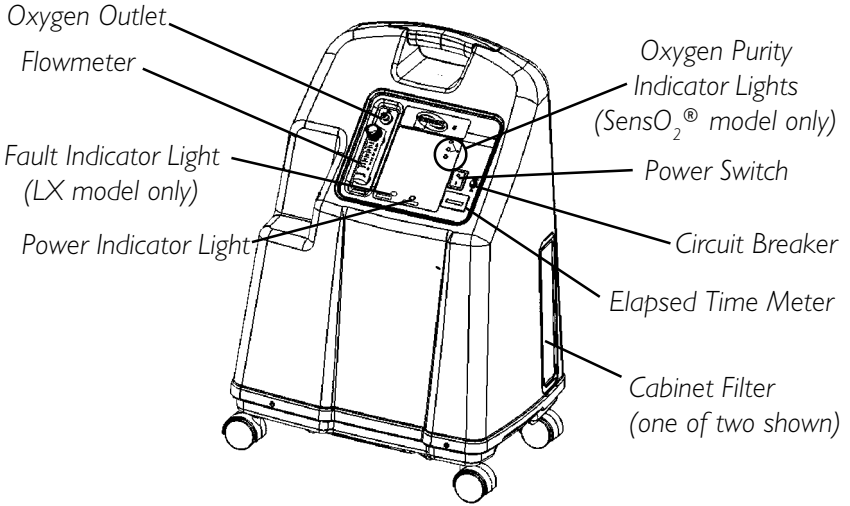

REAR VIEW

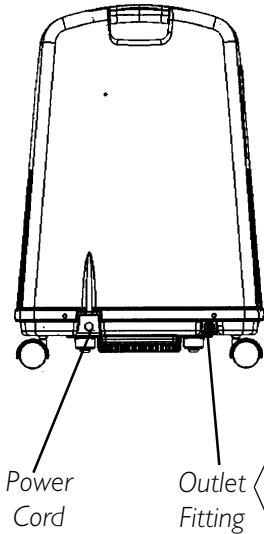ACCESSORIES (NOT SHOWN):

Remote Humidifier/Flowmeter
 Accessory - IRCRF5

PreciseR_x[™] Pediatric Humidifier/
 Flowmeter Accessory - IRCPF16

Venture[®] Homefill[™] II home
 oxygen compressor - IOH200

NOTE: This outlet fitting is to be used only for filling oxygen cylinders with the Venture HomeFill II home oxygen compressor. The outlet fitting does not affect concentrator performance. Refer to the HomeFill II owner's manual, part number 1100873, for connection and operating instructions. When not in use, the plug provided with the concentrator should be inserted into the outlet fitting. For more information about the HomeFill II, contact your Invacare dealer.

SPECIFICATIONS

SPECIFICATIONS

Electrical Requirements: 120 VAC \pm 10% (132 VAC/108 VAC),
60 Hz

Rated Current Input: 4.0 A

Sound Level: 50 dB Average

Altitude: Up to 6,000 ft. (1828 meters) above sea level without degradation of concentration levels. From 6,000 ft. (1828 meters) to 13,129 ft. (4000 meters) below 90% efficiency.

***Oxygen Output Concentration Levels:** ALL 5LXO2/5LX MODELS
95.6% to 87% at 1/2 to 5 L/min.

*** Stated concentration levels achieved after initial warm-up period (approximately 30 minutes).**

Maximum Outlet Pressure: 5 ± 0.5 psi.
(34.5 ± 3.45 kPa.)

SensO₂[®] Alarm Thresholds:

Label Symbol	O ₂ Purity	Indicators
	Below 73%	RED Indicator Light - Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown
	73% to 85%	YELLOW Indicator Light
O ₂	Over 85%	GREEN Indicator Light

SPECIFICATIONS (CONTINUED)

Flow Range:	1/2 - 5 L/min. For flowrates less than 1 L/min., we recommend the use of the Invacare Pediatric Flowmeter IRCPF16.
Average Power Consumption:	400W
Pressure Relief Mechanism Operational at:	35 psi \pm 3.5 psi (24.1 kPa \pm 24.1 kPa)
Change in maximum recommended flow when back pressure of 7kPa is applied:	0.7 L/min.
Filters:	Cabinet (2), Outlet HEPA and Compressor Inlet.
Safety System:	Current overload or line surge shutdown. High temperature compressor shutdown. High Pressure Alarm w/compressor shutdown. Low Pressure Alarm w/compressor shutdown. Battery Free Power Loss Alarm. SensO ₂ [®] Oxygen System (SensO₂[®] Model)
Width:	18 3/8" \pm 3/8" (46.7 cm \pm 1 cm)
Height:	26 3/8" \pm 3/8" (67.0 cm \pm 1 cm)
Depth:	14 3/8" \pm 3/8" (36.5 cm \pm 1 cm)
Weight:	51 lbs. \pm 1 lb (23.2 kg \pm .5 kg)
Shipping Weight:	56 lbs. \pm 1 lb (25.4 kg \pm .5 kg)

SPECIFICATIONS

SPECIFICATIONS (CONTINUED)

Operating Ambient

Temperature: 50°F - 95°F (10°C - 35°C) at 20-60% relative humidity

Exhaust: Less than Ambient + 35°F (+19°C).

Oxygen Output: Less than Ambient + 4°F (+2°C).

Cabinet: Impact Resistant flame-retardant plastic cabinet that conforms to UL 94-V0.

Regulatory Listing: DOUBLE INSULATED PRODUCT
ETL certified complying with UL 1097 (U.S. models).
CSA certified to C22.2 No. 68 (Canadian models).

Electrical: No extension cords.

Placement: No closer than 3-inches from any wall, furniture, draperies, or similar surfaces.

Tubing: Crush-Proof Tubing - 7 ft. (2.1m) cannula with a maximum 50 ft. (15.2 m) extension **(DO NOT PINCH)**.

Relative Humidity: 20 to 60%

Time of Operation: Up to 24 hours per day.

Recommended Storage and Shipping Temperature: -20°F to 150°F (-29°C to 65°C),
at 15-95% relative humidity

HANDLING

The concentrator should always be kept in the upright position to prevent cabinet damage while being transported. The shipping container has been designed to assure maximum protection of the concentrator. If the concentrator is to be reshipped by common carrier, additional cartons are available from Invacare.

UNPACKING

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the Carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

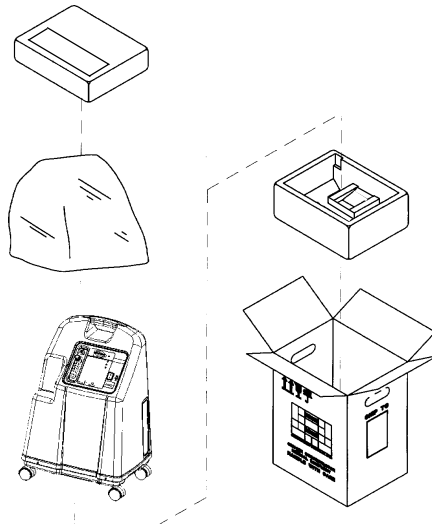
FIGURE I - PACKAGING

NOTE: Unless the Oxygen Concentrator is to be used immediately, retain containers and packing materials for storage until use of the concentrator is required.

INSPECTION

1. Examine exterior of the Oxygen Concentrator for nicks, dents, scratches or other damages. Inspect all components.

STORAGE

1. Store the repackaged Oxygen Concentrator in a dry area.
2. **DO NOT** place other objects on top of the repackaged concentrator.

INTRODUCTION/ OPERATING INSTRUCTIONS

INTRODUCTION

Your Oxygen Concentrator is intended for individual use in the home. It is an electronically operated device that separates oxygen from room air. It provides high concentration of oxygen directly to you through a nasal cannula or mask. Clinical studies have documented that oxygen concentrators are therapeutically equivalent to other types of oxygen delivery systems.

Your Provider will show you how to use your oxygen concentrator. He/She should be contacted with any questions or problems regarding your Oxygen Concentrator. This Owners Manual will tell you about your concentrator and will serve as a reference as you use your concentrator.

OPERATING INSTRUCTIONS

SELECT A LOCATION

You may select a room in your house where using your oxygen concentrator would be most convenient. Your concentrator can be easily rolled from room to room on its casters.

Your Oxygen Concentrator will perform best when operated under the following conditions. Usage in environments other than those described may result in the need for increased equipment maintenance. The air intake of the unit should be located in a well ventilated area to avoid airborne pollutants and/or fumes.

RECOMMENDED END-USER GUIDELINES FOR OPTIMUM PERFORMANCE:

Temperature:	50°F - 95°F (10°C - 35°C)
Electrical:	No extension cords.
Placement:	No closer than 3-inches from the wall, furniture, draperies, or similar surfaces.
Tubing and Cannula:	7 ft. cannula with a maximum 50 ft. of Crush-Proof Tubing (DO NOT PINCH).
Environment:	Smoke and soot-free. No confined spaces (Example: NO closets).
Relative Humidity:	20 to 60%
Time of Operation:	Up to 24 hours per day.
Flow Rate	From 1/2 L/min. to 5 L/min. For flow rates less than 1 L/min., use the Pediatric Flowmeter.
Minimum Operating Time:	30 Minutes

OPERATING INSTRUCTIONS

NOTE: Ensure that your concentrator is at least (3) three-inches away from walls, draperies or furniture to assure sufficient air flow. Avoid deep pile carpets and heaters, radiators or hot air registers.

PLUG IN POWER CORD

1. Plug in power cord to an electrical outlet.

CONNECT HUMIDIFIER (IF SO PRESCRIBED)

1. Fill humidifier with distilled water to the level indicated by the manufacturer.

WARNING

DO NOT overfill humidifier.

Replace the humidifier cap and securely tighten.

DO NOT reverse the oxygen input and output connections.

Water from the humidifier bottle **WILL** travel through the cannula back to the patient.

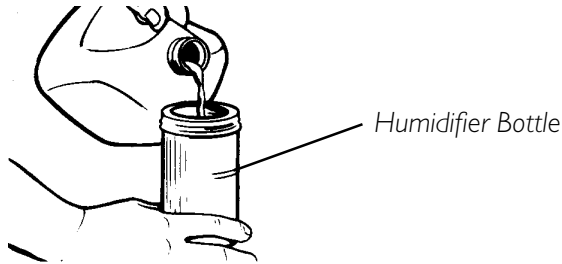
FIGURE 1 - FILLING THE HUMIDIFIER

2. Place the humidifier in the humidifier compartment.

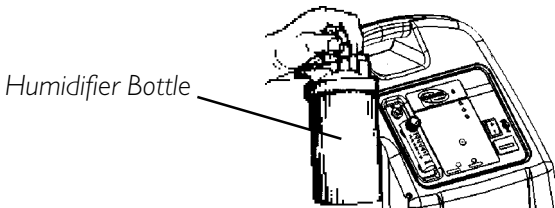
FIGURE 2 - HUMIDIFIER COMPARTMENT

OPERATING INSTRUCTIONS

3. Remove the humidifier adapter from its stored location inside the filter access door and attach it to the humidifier by turning the wing nut on the humidifier bottle counterclockwise until it is securely fastened.

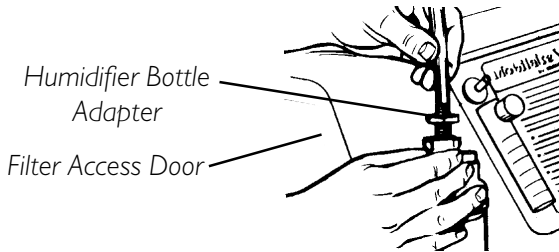
FIGURE 3 - ATTACHING THE HUMIDIFIER ADAPTER

NOTE: If using the Remote Humidifier/Flowmeter Accessory, refer to the Operating Instructions which accompany the accessory.

4. Attach oxygen tubing from the humidifier bottle to oxygen outlet connector on the oxygen concentrator.

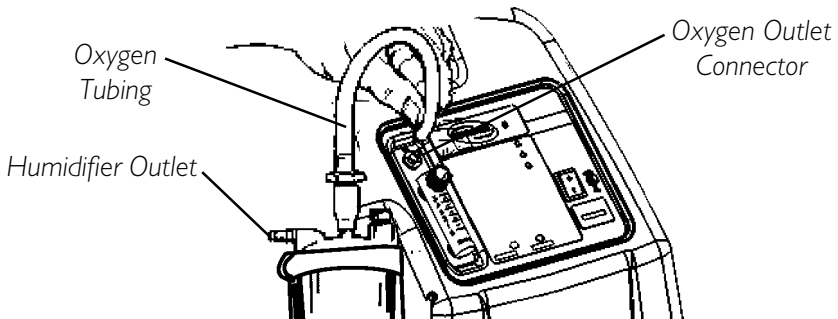
FIGURE 4 - ATTACHING THE OXYGEN TUBING

5. Attach the cannula/patient supply tubing to the humidifier bottle outlet.
6. After assembly, ensure that oxygen is flowing through the cannula.

WARNING

For optimum performance, Invacare recommends that concentrator be operated a minimum of 30 minutes during any use. Shorter periods of operation may adversely affect the life of the product.

POWER SWITCH

1. Press power switch to the **ON** position. The power light (I/O), located on the control panel, will illuminate a **GREEN** light. The alarm will "beep" three (3) times indicating that the system is functioning properly (FIGURE 5).

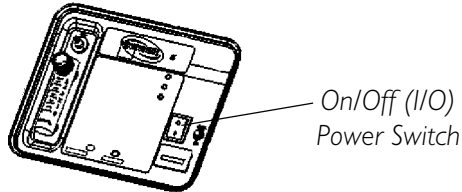
FIGURE 5 - POWER SWITCH


FLOWRATE

1. Turn the flowrate knob to the setting prescribed by your physician or therapist.

WARNING

DO NOT change the L/min. setting on the flowmeter unless a change has been prescribed by your physician or therapist.

NOTE: To properly read the flowmeter, locate the prescribed flowrate line on the flowmeter. Next, turn the flow knob until the ball rises to the line. Now, center the ball on the L/min. line prescribed (FIGURE 6).

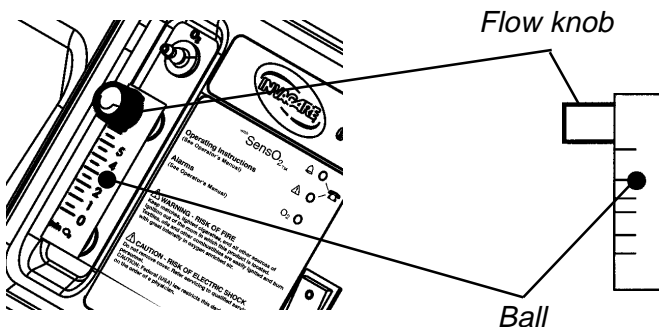**FIGURE 6 - ADJUSTING THE FLOWRATE-
PLATINUM 5 WITH SENSO₂[®] SHOWN**

OPERATING INSTRUCTIONS

SENSO₂[®] OXYGEN PURITY INDICATOR- IF YOUR UNIT FEATURES THE O₂ SENSOR FEATURE

This feature monitors the purity of oxygen generated by the oxygen concentrator. If purity falls below factory preset standards, indicator lights on the control panel will illuminate.

INITIAL STARTUP OF THE CONCENTRATOR

NOTE: Concentrator may be used during the initial start warm-up time (approximately 30 min.) while waiting for the GREEN O₂ purity light to illuminate.

When the unit is initially turned on, the **GREEN** power LED (I/O) will illuminate. Within 5 to 30 minutes after initial startup, a **GREEN** light will illuminate in the **Oxygen Purity** section of the **Control Panel (FIGURE 7)** if concentration is within specification. Any other SensO₂[®] indicator lights illuminating will signal a condition that requires a call to the supplier.

EXPLANATION OF OXYGEN PURITY INDICATOR LIGHTS (FIGURE 7):

GREEN LIGHT (O₂) - Normal Operation

YELLOW LIGHT (⚠) - Call Supplier **IMMEDIATELY**. You may continue to use the concentrator unless instructed otherwise by your supplier. Be certain that **BACKUP OXYGEN** is nearby.

RED LIGHT (🔔) - Total unit shutdown. Switch **IMMEDIATELY** to **BACKUP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

GREEN LIGHT - with YELLOW LIGHT FLASHING - Call Supplier **IMMEDIATELY**. Oxygen sensor malfunctioning; you may continue to use the concentrator.

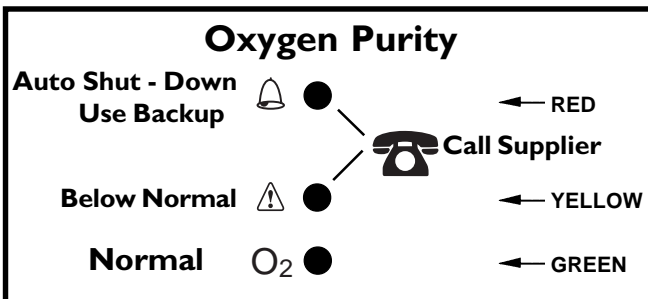
FIGURE 7 - O₂ DISPLAY CONTROL PANEL

OXYGEN PURITY INDICATOR- IF YOUR UNIT DOES NOT FEATURE THE O₂ SENSOR

LX MODELS INDICATOR LIGHT EXPLANATION (FIGURE 8):

RED LIGHT (⚠) - Total Unit Shut-Down. Switch **IMMEDIATELY** to a **BACK-UP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

GREEN LIGHT (I/O)- On/Off.

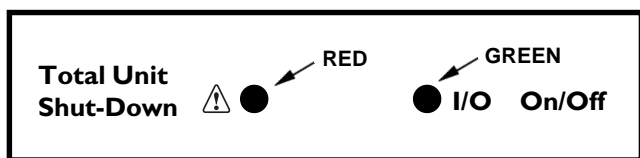
FIGURE 8 - DISPLAY CONTROL PANEL

MAINTENANCE

CLEANING THE CABINET FILTERS

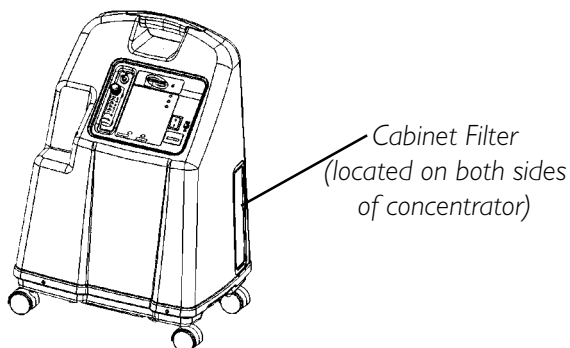
FIGURE 9 - CABINET FILTER LOCATION

NOTE: There are two (2) cabinet filters. One (1) is located on each side of the cabinet (FIGURE 9).

1. Remove each filter and **CLEAN AT LEAST ONCE A WEEK** depending on environmental conditions (**FIGURE 10**).


NOTE: Environmental conditions that may require more frequent cleaning of the filters include but are not limited to: high dust, smoking, air pollutants, etc.

MAINTENANCE

2. Clean the cabinet filters with a vacuum cleaner or wash in warm soapy water and rinse thoroughly (**FIGURE 10**).
3. Dry the filters thoroughly before reinstallation.

CAUTION

DO NOT operate the concentrator without the filters installed.

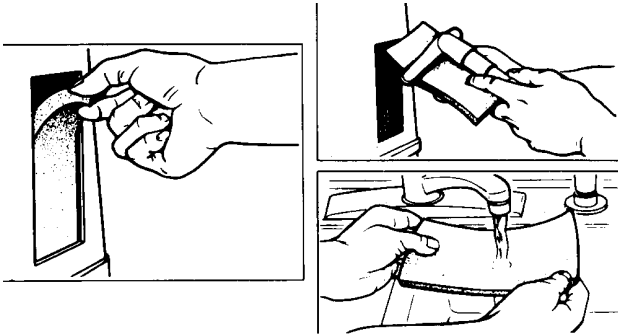**FIGURE 10 - CLEANING THE CABINET FILTERS
CABINET**

WARNING

UNPLUG the concentrator when cleaning. To avoid electrical shock, **DO NOT** remove cabinet.

1. Clean the cabinet with a mild household cleaner and non-abrasive cloth or sponge.

HUMIDIFIER

NOTE: To clean the oxygen humidifier, follow the instructions provided by the manufacturer. If none are provided, follow these steps:

1. Clean the humidifier every day.
2. Wash it in soapy water and rinse it with a solution of ten (10) parts water and one (1) part vinegar.
3. Rinse thoroughly with hot water and refill with distilled water to the level shown on the humidifier.

WARNING

DO NOT overfill Humidifier Bottle.

TROUBLESHOOTING GUIDE

SYMPTOM:	PROBABLE CAUSE:	SOLUTION:
<p>Alarm: SHORT BEEPS, LONG PAUSE</p> <p>Concentrator NOT operating, power switch ON.</p> <p>Beep..... Beep.....</p>	<p>Main Power Loss:</p> <ol style="list-style-type: none"> 1. Power cord NOT plugged In. 2. No power at outlet. 3. Tripped circuit breaker. 	<ol style="list-style-type: none"> 1. Insert plug into outlet. 2. Inspect house circuit breakers or fuses. If problem recurs, use a different outlet. 3. Call Provider for Service.
<p>Alarm: CONTINUOUS Power Switch ON,</p> <p>Concentrator NOT OPERATING.</p> <p>Beeeeeeeep....</p>	<p>System Failure:</p> <ol style="list-style-type: none"> 1. Unit overheating due to blocked air intake. 2. Insufficient power at outlet. 3. Internal repairs required. 	<ol style="list-style-type: none"> 1a. Remove and clean cabinet filters. 1b. Move oxygen concentrator at least three (3) inches away from walls, draperies or furniture. 2. Do not use extension cords. Move to another electrical outlet or circuit. 3. Call Provider for Service.
<p>YELLOW or RED LIGHT Illuminated.</p> <p>* Only applies to SensO₂ models.</p>	<ol style="list-style-type: none"> *1. Low oxygen purity. *2. Kinked or blocked tubing, cannula or humidifier. *3. Flowmeter set at 1/2 L/min. or less. 4. Unit overheating due to blocked air intake. 5. Insufficient power at outlet. 6. Internal repairs required. 	<ol style="list-style-type: none"> 1. Clean or Replace filters. 2. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn power back ON. 3. Check flowmeter is set to 1.0 L/min. or more. Refer to Recommended End-User Guidelines in the Operating Instructions of this manual (page 12). 4a. Remove and clean cabinet filters. 4b. Move concentrator at least 3- inches from walls, draperies, and furniture. 5. Do not use extension cords. Move to another electrical outlet or circuit. 6. Call Provider for Service.
<p>GREEN LIGHT with YELLOW Light Flashing.</p>	<ol style="list-style-type: none"> 1. Internal repairs required. 	<ol style="list-style-type: none"> 1. Call Provider for Service.

PREVENTIVE MAINTENANCE RECORD INVACARE PLATINUM OXYGEN CONCENTRATOR

Yes, you can:

ON EACH INSPECTION

Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				
EVERY 90 DAYS																				
Check Oxygen Concentration (UNITS WITHOUT SensO ₂ ®)																				
ANNUALLY OR DURING PREVENTIVE MAINTENANCE SCHEDULE																				
Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter*																				
Check Compressor Inlet Filter*																				
Check Oxygen Concentration (SensO ₂ ® MODELS)																				
Check Power Loss Alarm																				
AS REQUIRED																				
Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				

MODEL NO. IRC _____ SERIAL NO. _____

PREVENTIVE MAINTENANCE RECORD

*Refer to Preventive Maintenance section of Service Manual.

LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to original purchaser of our products. This warranty gives you specific legal rights and you may also have other legal rights which may vary from state to state.

Invacare Corporation warrants its Platinum Oxygen Concentrator to be free from defects in materials and workmanship from date of purchase for a period of five (5) years

The 4-way valve body carries a lifetime warranty from date of purchase. **DO NOT OPEN OR ATTEMPT TO SERVICE VALVE BODY.** This will void any and all warranty applicable to the valve body. If within such warranty period any such product shall be proven to Invacare Corporation's satisfaction to be defective, such product shall be repaired or replaced, at Invacare Corporation's option. This warranty only applies to the labor for repairs performed by the Invacare Service Department or authorized Invacare Authorized Service Centers. It does not apply to the labor performed by the purchaser or user. This warranty does not include normal wear and tear or shipping charges. Invacare Corporation's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Routine maintenance items, such as filters, are excluded from this warranty.

For warranty service, please contact Invacare Corporation's service department at the toll free number on the back cover.

Upon receiving notice of an alleged defect in a product, Invacare Corporation will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return the entire unit or remove, at purchaser's cost, the defective component part(s) identified, pack the component part(s) in a manner to avoid shipping damage and to ship the component part(s) to either Invacare Corporation's plant or service center as specified by Invacare Corporation in advance. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. Do not return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

WARRANTY

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN, AND THE REMEDY FOR VIOLATION OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE CORPORATION SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Some states do not allow the exclusion or limitation of incidental or consequential damage, or limitation of how long an implied warranty lasts, so the above exclusion and limitation may not apply to you.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2001 Invacare Corporation
Invacare, SensO₂, and Venture are registered trademarks of Invacare Corporation.
Homefill, Sieve-Gard, and Yes, you can. are trademarks of Invacare Corporation.